Prior to use, carefully read the instructions for use.
 **(EN)**

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### DESCRIPTION

GC MI Paste Plus is a water based creme containing RECALDENT™ \* with incorporated fluoride (CPP-ACPF; Caséin Phosphopéptide-Amorphous Calcium Phosphate Fluoride). The level of fluoride is 0.2% w/w (900ppm), which approximates that in adult strength toothpastes. When CPP-ACPF is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium, phosphate and fluoride.

### IMPORTANT :

Saliva will enhance the effectiveness of CPP-ACPF and the flavour will help stimulate saliva flow. The longer CPP-ACPF and saliva are maintained in the mouth, the more effective the result.

**A topical paste containing bio-available calcium, phosphate and fluoride**

- Provides extra protection for teeth.
- Helps neutralize acid challenges from acidogenic bacteria in plaque.
- Helps neutralize acid challenges from other internal and external acid sources.

### CONTRAINDICATIONS

Caséin phosphopéptide are derived from milk caséin. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzozate preservatives.

### DIRECTIONS FOR USE INDICATIONS

- Following in-office bleaching procedures.
  - After ultrasonic, hand scaling or root planing.
  - Following professional tooth cleaning (P.T.C.).
  - Hypersensitivity prevention and control.
- As an alternative means of applying fluoride topically in children aged 6 and above.
  - During orthodontic treatment.
  - For high risk caries patients.
  - As a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
- For special needs adult patients.

### In-office application:

- Custom tray application**
  - Before use rinse the custom tray thoroughly under running water.
  - Extrude a generous layer of GC MI Paste Plus into the tray and apply to the upper and / or lower teeth.
  - Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
  - Remove the tray.
- Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1- 2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
- Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

### II. Non tray application:

- For easy, easy, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
- Apply GC MI Paste Plus to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
- Leave GC MI Paste Plus undisturbed for a minimum of 3 minutes.
- Then instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

### At-home application:

#### III. Day time application after tooth brushing as recommended by a dental professional

- Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
- Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
- Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the result.
- Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

### IV. Night time application after tooth brushing as recommended by a dental professional

The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.

- Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
- Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
- Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

### FLAVOURS

Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)

### STORAGE

Store in a cool dry/low humidity area away from direct sunlight (8-25°C) (46.4-77.0°F).

(Shelf life: 2 years from date of manufacture)

### PACKAGES

- Tube containing 40g (35mL)
- 1 Assorted Package
  - 10 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)
- Single Flavour Package
  - 10 x 40g in each flavour
- Vanilla, Strawberry\*\*, Melon\*\*, Tutti-Frutti\*\*, Mint\*\*
- \*\*available in Europe only

### Refill Package

- 1 x 40g tube in each flavour

### NOTE

GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.
\*RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

### CAUTION

- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
- It is not recommended to use GC MI Paste Plus on patients under the age of 6.
- The overnight at home application is not recommended for children under 12.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of spillage on clothes, wash off with water.
- If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzozate preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician.

Last revised : 07/2008

MANUFACTURED BY
**GC CORPORATION**
76-1 Hasunuma-cho, Tabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
**GC CORPORATION**
76-1 Hasunuma-cho, Tabashi-ku, Tokyo 174-8585, Japan

**EU: GC EUROPE N.V.**
Research-Park, Interleuvenlaan 13, B-3001 Leuven, Belgium
TEL: +32 16 39. 80. 50

**GC AMERICA INC.**
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1 708 507 0900

**GC ASIA DENTAL PTE. LTD.**
19,Loyang Way, #05-27 Singapore 508724
TEL: +65 6546 7588

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Prior to use, carefully read the instructions for use.

**EN**

## GC MI Paste Plus

TOPICAL CREME WITH CALCIUM, PHOSPHATE AND FLUORIDE

For use by or under the supervision of a dental professional in the recommended indications.

Use by or under the supervision of a dental professional in the recommended indications.

**DESCRIPTION**
GC MI Paste Plus is a water based creme containing RECALDENT™ with incorporated fluoride. Contains phosphatidyl-Aminohydroxyapatite Calcium Phosphate Fluoride). The level of fluoride is 0.2% w/w (900ppm), which approximates that in adult toothpaste. When used (900ppm) is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bi-avaliable calcium, phosphate and fluoride.

**IMPORTANT:**
Saliva will enhance the effectiveness of CPP-ACPF and the flavour will help stimulate saliva flow. The longer CPP-ACPF and saliva are maintained in the mouth, the more effective the result.

**Topical paste containing bio-avaliable calcium, phosphate and fluoride**
1. Provides extra protection for teeth.
2. Helps neutralize acid challenges from adogenic bacteria in plaque.
3. Helps neutralize acid challenges from other internal and external acid sources.

**CONTRAINDICATIONS**
Casesin phosphoglycopates are derived from milk casen. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzate preservatives.

**DIRECTIONS FOR USE**
**INDICATIONS**
1. Patients with office bleaching procedures.
2. After ultrasonic, hand scaling or root planing.
3. Following professional tooth cleaning (P.T.C.)
4. To reduce plaque accumulation and prevent tartar build-up.
5. As an alternative means of applying fluoride topically in children aged 6 and above.
6. During orthodontic treatment.
7. For high risk caries patients.
8. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
9. For special needs adult patients.

**In-office application:**
1. Prepare using the custom tray thoroughly under running water.
2. Extrude a generous layer of GC MI Paste Plus into the tray and apply to upper and/or lower arches using a clean finger or applicator.
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1- 2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
7. Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

**Non tray application:**
1. Apply a generous amount of excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply GC MI Paste Plus to the tooth surfaces using an application swab, finger or applicator. Avoid spreading the remaining GC MI Paste Plus into the tongue.
3. Leave GC MI Paste Plus undisturbed for a minimum of 3 minutes.
4. Then repeat the procedure using a clean finger or applicator.
5. Leave GC MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

**At-home application:**
**Day time application after tooth brushing as recommended by a dental professional**
1. Apply a generous amount of GC MI Paste Plus to upper and lower teeth.
2. A pea size amount for each arch is the minimum required.
3. The product should be applied to the tooth surfaces using a clean dry finger or cotton tip.
4. For difficult areas (between the teeth) use an Interproximal Cleaning Brush.
5. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
6. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (further 1- 2 minutes) avoiding expectoration (spitting out) and delaying swallowing.
7. The longer GC MI Paste Plus and saliva are maintained in the result.
8. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

**Night time application after tooth brushing as recommended by a dental professional**
Note:
Overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
1. Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth.
2. A pea size amount for each arch is the minimum required.
3. The product should be applied to the tooth surfaces using a clean dry finger or cotton tip.
4. For difficult areas (between the teeth) use an Interproximal Cleaning Brush.
5. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
6. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
7. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

**FLAVOURS**
Strawbery (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)
**STORAGE**
Store in a cool, dry low humidity area away from direct sunlight (8-25°C / 46-77.0°F). (Shelf life: 2 years from date of manufacture)

**PACKAGES**
Tube containing 40g (35mL)
1. Assorted Package
10 x 40g (no 2) of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti
3. Single Flavour Package
10 x 40g in each flavour
Vanilla Strawberry, Melon, Tutti-Frutti, Mint
3. Refill Package
1 x 40g tube in each flavour

**NOTE**
GC MI Paste Plus with its unique CPP-ACPF formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia. RECALDENT is a trademark used under licence. CPP-ACPF technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

**CAUTION**
1. Always replace the cap after use and remove any residual paste or bacteria from the end of the applicator.
2. It is not recommended to use GC MI Paste Plus on patients under the age of 6.
3. The overnight at home application is not recommended for children under 12.
4. In case of contact with eyes, wash immediately with water and seek medical attention.
5. In case of spillage on clothes, wash off with water.
6. If any allergic reactions occur, discontinue use. Symptoms may indicate sensitivity or allergy to the benzate preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician.

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**III. Night time application after tooth brushing as recommended by a dental professional**
Note:
Overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
1. Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth.
2. A pea size amount for each arch is the minimum required.
3. The product should be applied to the tooth surfaces using a clean dry finger or cotton tip.
4. For difficult areas (between the teeth) use an Interproximal Cleaning Brush.
5. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
6. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
7. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

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**III. Night time application after tooth brushing as recommended by a dental professional**
Note:
Overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
1. Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth.
2. A pea size amount for each arch is the minimum required.
3. The product should be applied to the tooth surfaces using a clean dry finger or cotton tip.
4. For difficult areas (between the teeth) use an Interproximal Cleaning Brush.
5. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
6. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (further 1- 2 minutes) avoiding expectoration (spitting out) and delaying swallowing.
7. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
8. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

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5. Požaduje pacienta, aby vyplivl a pokud je to možné, nepřevracoval. Zbytky GC MI Paste Plus se postupně vytratí.
6. Doporučte pacientovi, aby 30 minut po aplikaci nejedl ani nepil.
**III. Night time application after tooth brushing as recommended by a dental professional**
Note:
Overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
1. Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth.
2. A pea size amount for each arch is the minimum required.
3. The product should be applied to the tooth surfaces using a clean dry finger or cotton tip.
4. For difficult areas (between the teeth) use an Interproximal Cleaning Brush.
5. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
6. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (further 1- 2 minutes) avoiding expectoration (spitting out) and delaying swallowing.
7. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
8. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

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