# COE-PAK™

## COE-PAK™ GB

### PERIODONTAL PASTE

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
*NO EUGENOL, NO ASBESTOS.* Prior to use, carefully read the instructions. For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS:

- Use Coe-Pak whenever a periodontal paste is required.
- No eugenol...will not burn sensitive mouth tissues or craze plastic restorations.
  - Has resilient hardness...will not fracture and leave sharp edges to lacerate tissue.
  - Completely stable...will not deteriorate during normal conditions of storage.
  - Smooth, cohesive, easy to form into desired shapes.
- ...remains clean in the mouth...no unpleasant taste.*

### DIRECTIONS FOR USE:

1. Use a spatula and a COE mixing pad. Do **not** use a rigid or narrow spatula. Thorough mixing is very important. This is more easily done with a wide, flexible spatula.
2. *When using COE-PAK AUTOMIX, load the cartridge into the dispenser and attach a mixing tip. Express the required length of COE-PAK onto the mixing pad. (Do not use more than 4 full strokes of the trigger). Leave the AUTOMIX mixing tip in place until required for the next use.*
3. For non-automix, extrude equal lengths of paste from catalyst and base tube. Spatulate for 30-45 seconds or until the two pastes are a uniform color. If you would like a harder dressing, add a small amount of zinc oxide powder to the pink paste (catalyst) before spatulating.
4. Using a spatula, form the mixed pastes into a cylindrical shape.

## COE-PAK™ D

# COE-PAK™

## COE-PAK™ D

5. Lightly lubricate your fingers with a thin film of vaseline, lanolin, cold cream, or water and test the mixed pastes for tackiness (it should lose its tackiness after 2-3 minutes; 1 minute for Hard & Fast set). **NOTE:** *COE-PAK AUTOMIX can be handled after about 30 seconds. Lubrication of fingers is not necessary.*
6. COE-PAK can be handled and molded 2 to 3 minutes after mixing. With your finger tips roll the cylindrical form off the COE mixing pad (1 minute for Hard & Fast).
7. COE-PAK remains workable for 10-15 minutes (5-8 minutes for Hard & Fast and at least 8 minutes for Automix). During this time it can be formed in the usual manner into ropes or other desired shapes.
8. During its working stage COE-PAK is extremely cohesive. It can be formed into ropes of virtually any diameter for easy placement against tissue and teeth.

<b>COMPARISON OF COE-PAK/COE-PAK HARD &amp; FAST/COE-PAK AUTOMIX</b> (equal lengths -1/2 minute mixing time)			
	COE-PAK REGULAR	COE-PAK HARD & FAST	COE-PAK AUTOMIX
After mixing, ready to use	3 minutes	1 minute	30 seconds
Consistency	plastic	heavy, but still plastic	plastic
Working time	10-15 minutes	5-8 minutes	at least 8 minutes
Ability to bond one piece of material to another	10-15 minutes	5-8 minutes	8-12 minutes
Final hardness	30 minutes (still resilient but not brittle)	10 minutes (much harder than COE-PAK, but not brittle)	10 minutes

**NOTE:** *Sensitivity to some of the components of this product has been reported. The reported sensitivity varies from simple dermal redness, to rash, swelling and nausea and/or vomiting. The practitioner should assure, **prior to use**, no such sensitivity to this product is present in the intended patient.*

The maximum shelf life of these products is 24 months from the manufacturing date.

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